



The Safest RF Lipoplasty

© 2010-2015 ★ StarLipo. All rights reserved

RF = Radio-Frequency

150 kHz - 4MHz

The StarLipo procedure consists in the intraoperative application of customized monopolar RF energy to achieve postoperative skin contraction.

The skin contraction effect starts immediately after the application of the monopolar RF energy. The maximum effect is achieved 6 months after surgery.

The StarLipo procedure is indicated in every case of primary liposuction to minimize postoperative skin flaccidity.

In most cases of mini-abdominoplasty, a StarLipo procedure can be offered as an alternative to mini-abdominoplasty.

Contraindications: secondary liposuction, deep scarring, and any pathological condition in the proposed operative area



StarLipo device



**Monopolar Radio-Frequency Antenna
with built-in RF conditioning circuitry**



The customized RF energy is delivered through the tip.

Do not use the StarLipo device for pre-tunneling because the StarLipo device is not a sturdy metal cannula. Starlipo device is a high-tech dedicated device

StarLipo technique



- 1) Preoperative marking
- 2) Perform a standard liposuction using metal cannulas as usual
- 3) Finally, use the Monopolar Radio-Frequency Antenna (StarLipo device) for skin contraction


1) Preoperative marking:
sketch several 2 cm wide
tunnels in a peacock tail
pattern




2) Perform a standard liposuction as usual

Do not use the StarLipo device for pre-tunneling because the StarLipo device is not a sturdy metal cannula. Starlipo device is a high-tech dedicated device

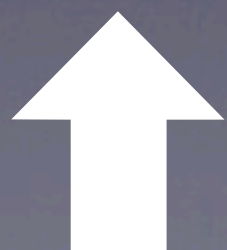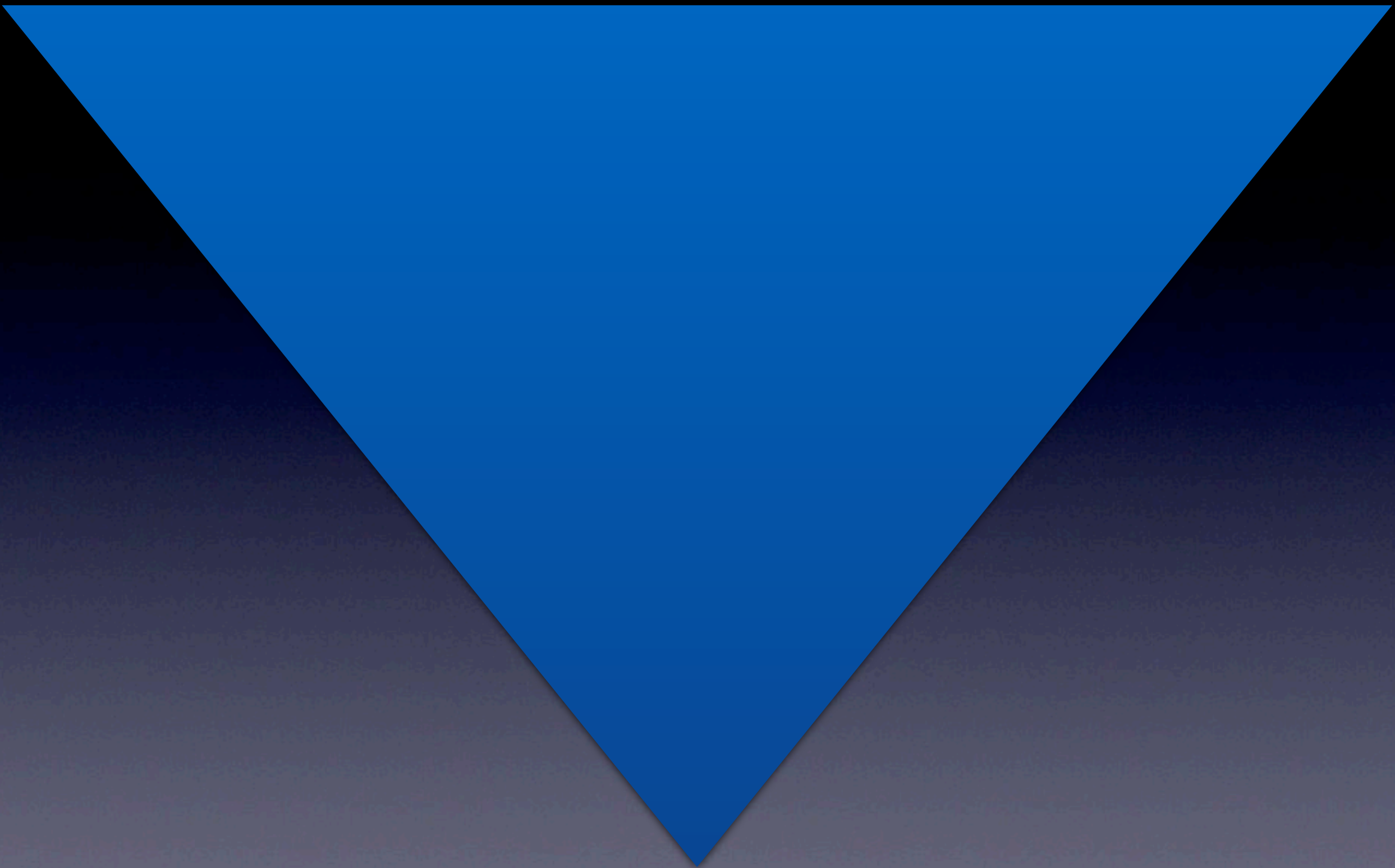

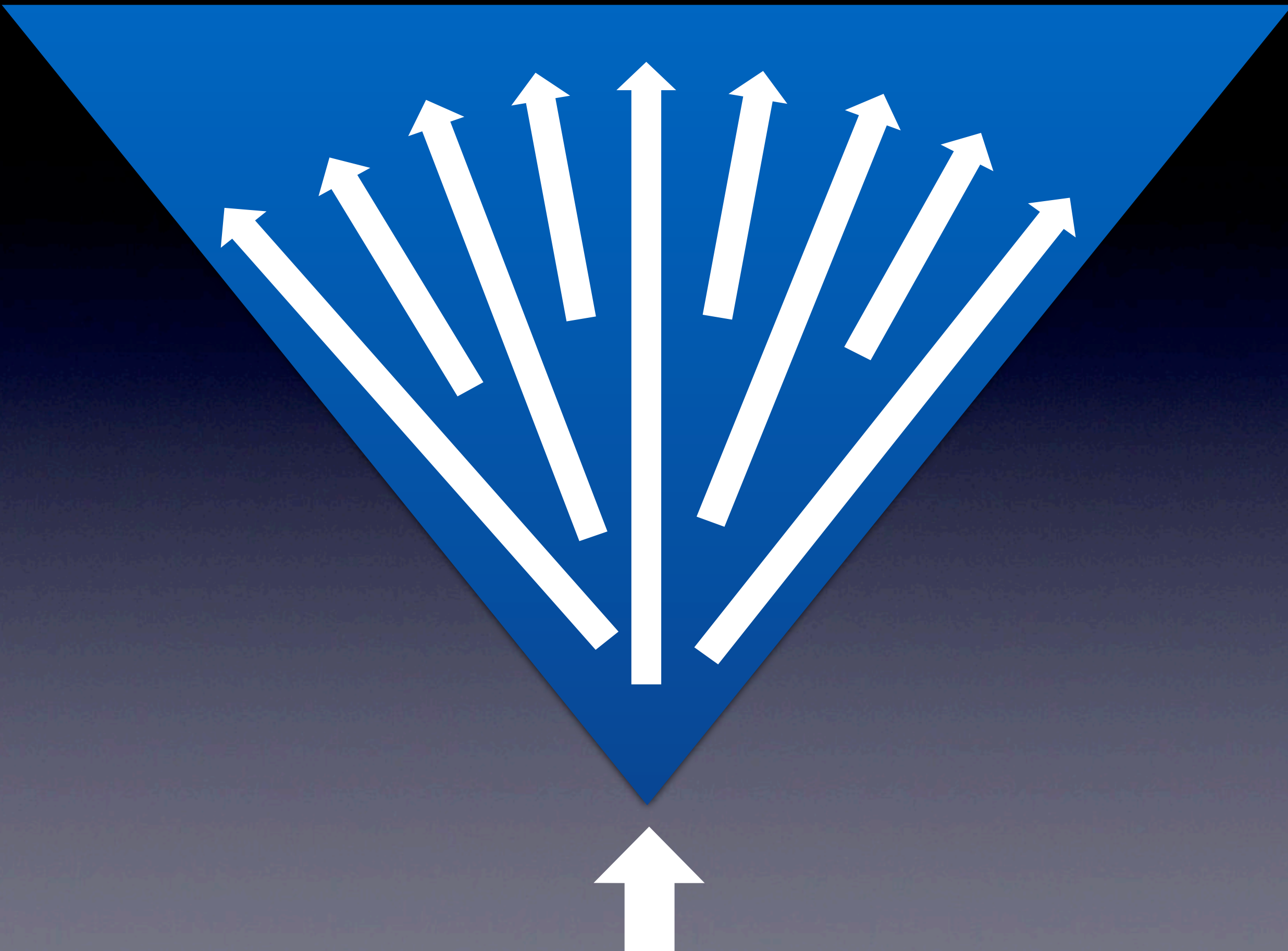

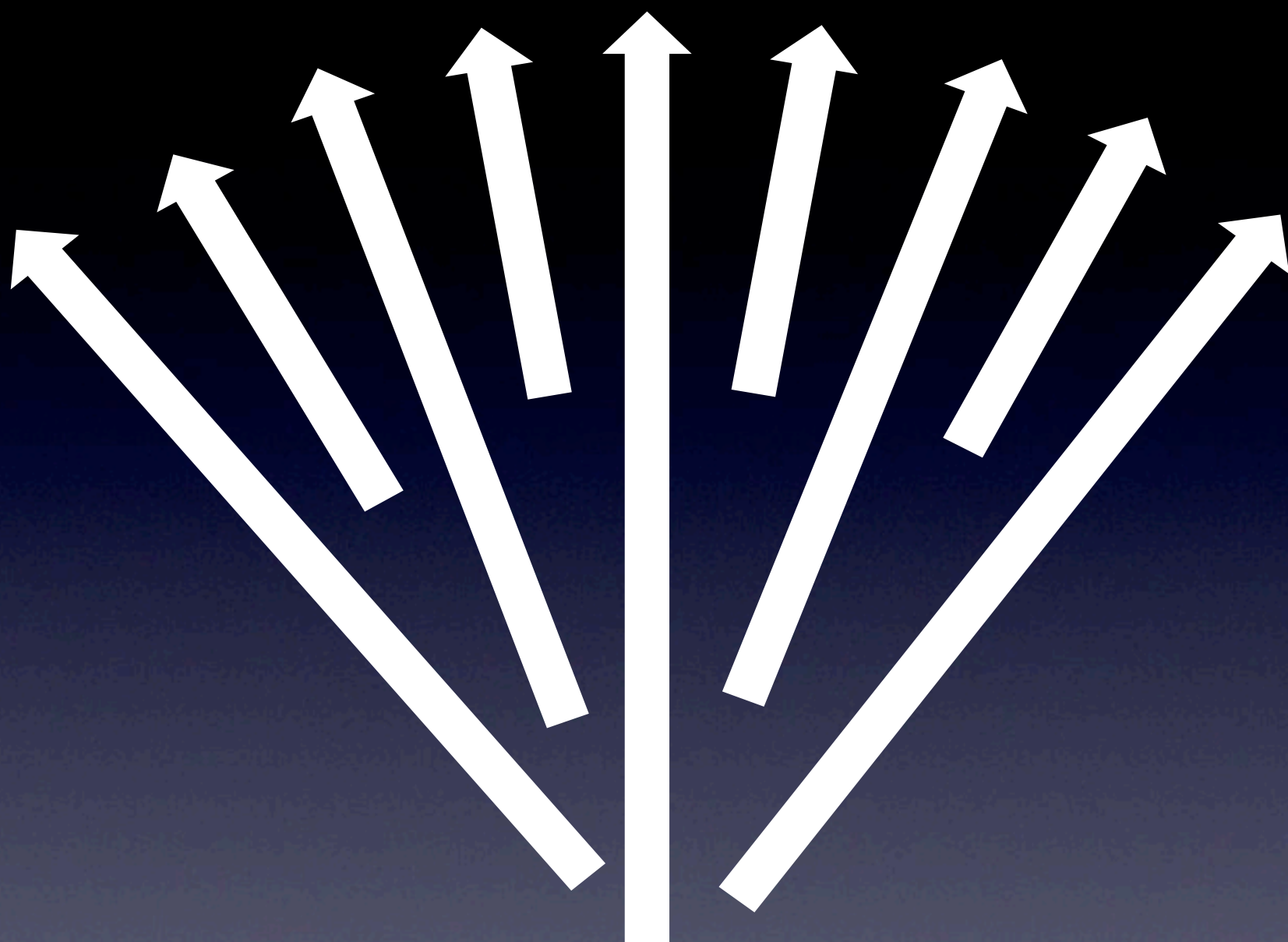



Once the liposuction
is completed, we can
proceed to the RF
application

3) Use the Monopolar Radio-Frequency Antenna (StarLipo device)

Connections




Set the
electrosurgical
unit at 30 watts
output,
coagulation



Double-check
the Patient
Return
Electrode is
firmly attached



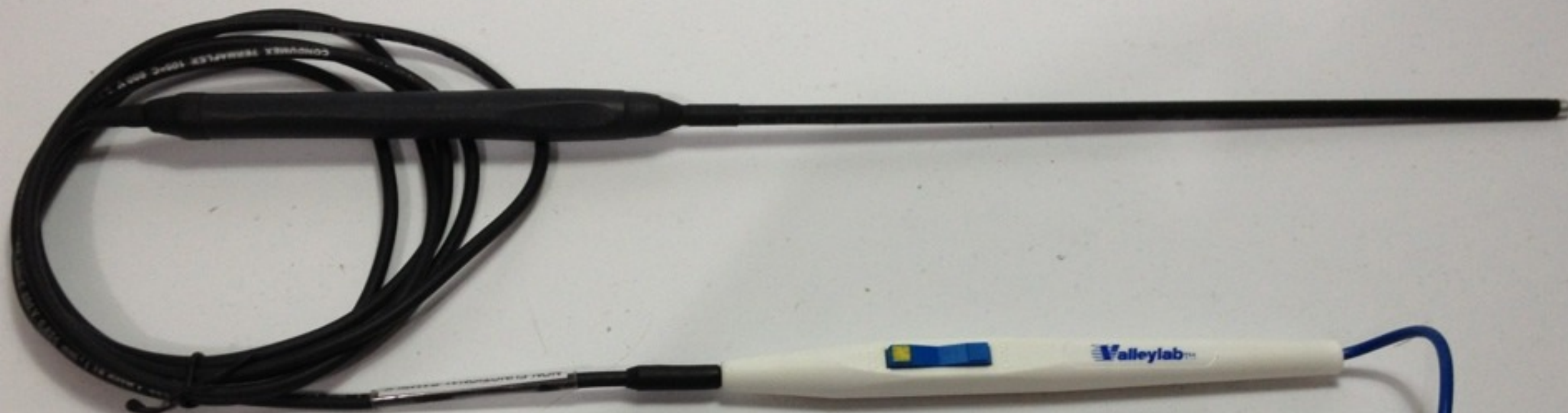




Radio-frequency energy application


1) Insert the StarLipo
through the liposuction
incision




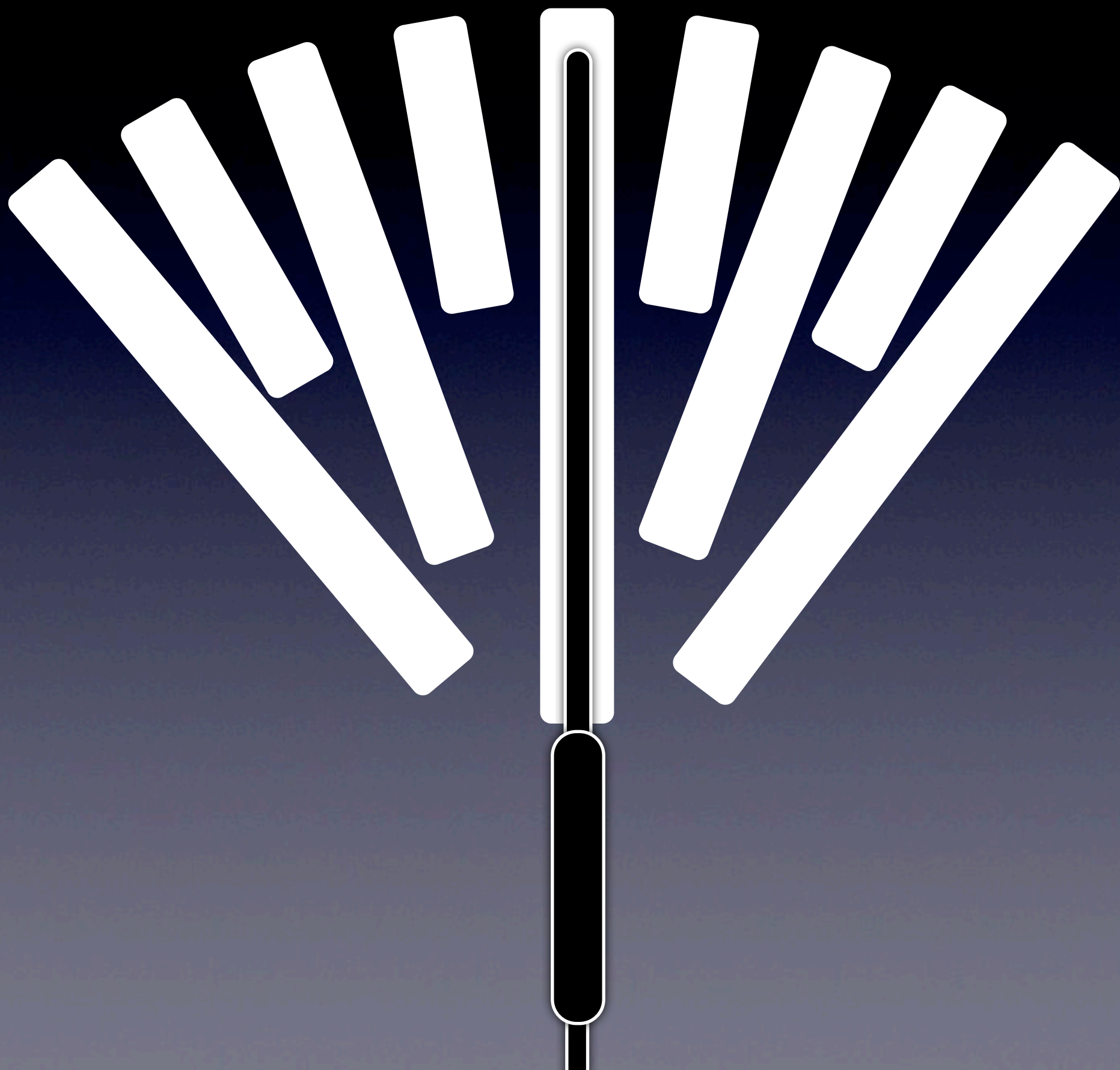



2) Press the electrosurgical unit (ESU) foot-switch (COAG mode) and slowly withdraw the StarLipo at a speed of 2 cm per second



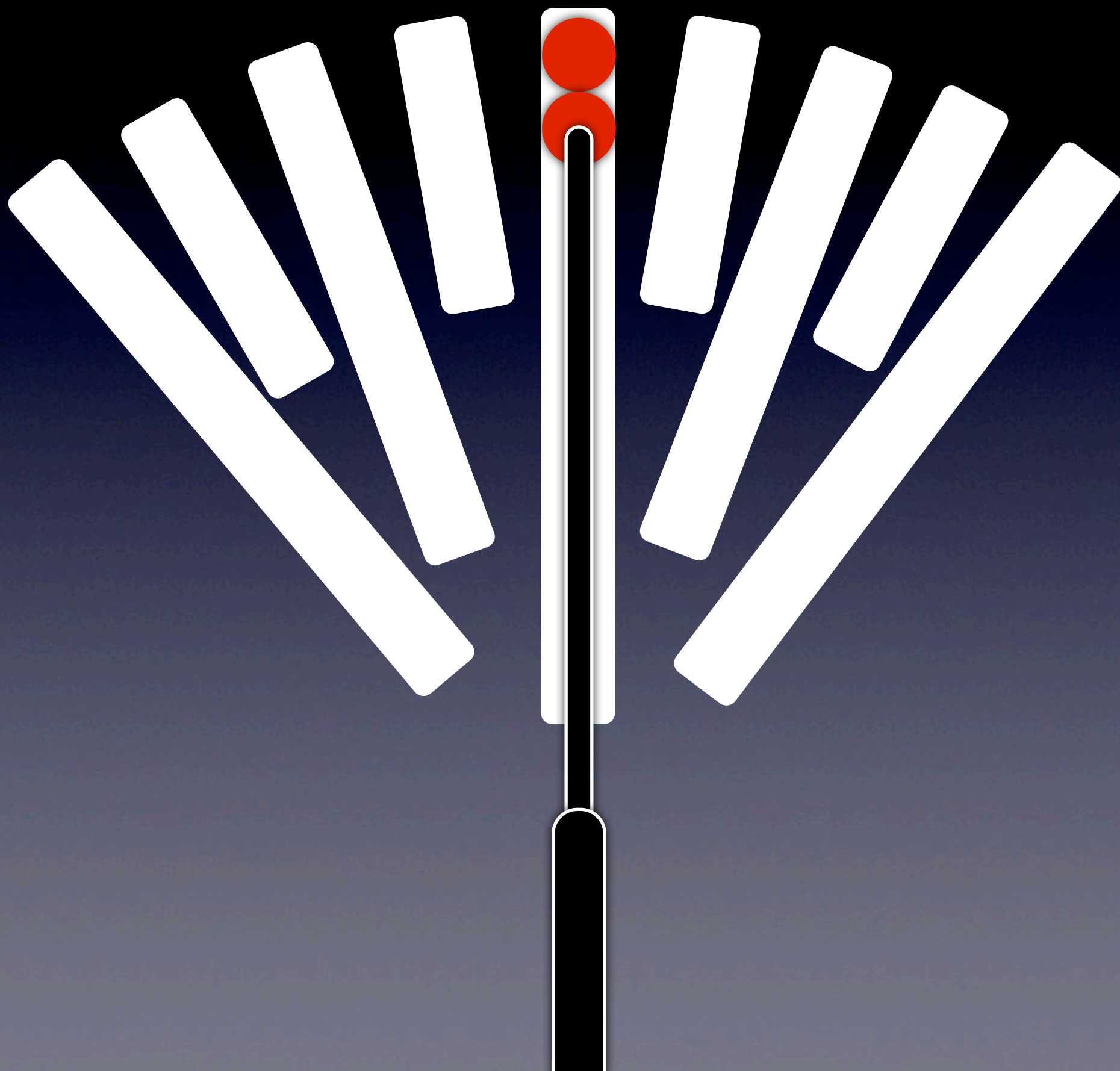


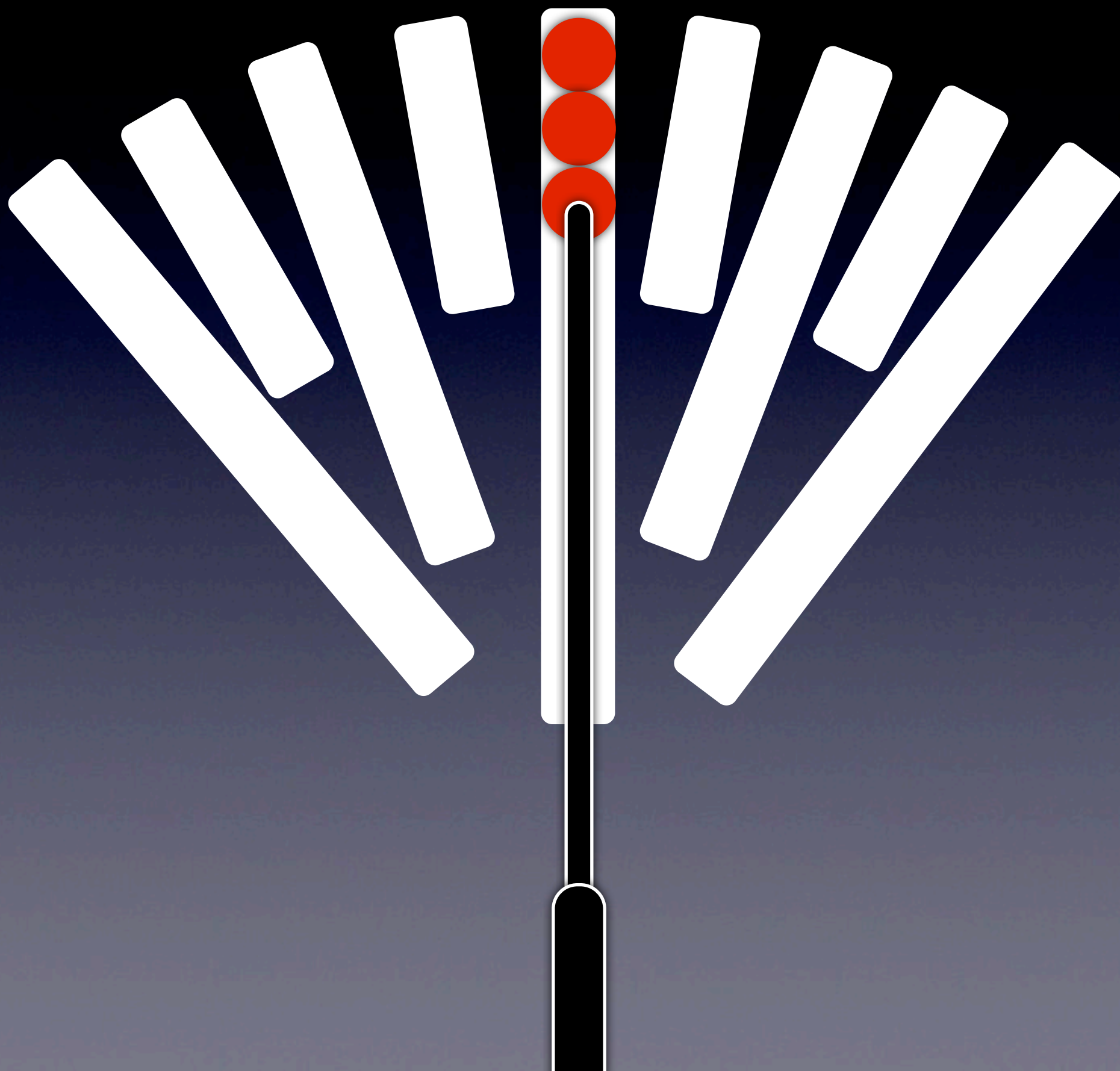

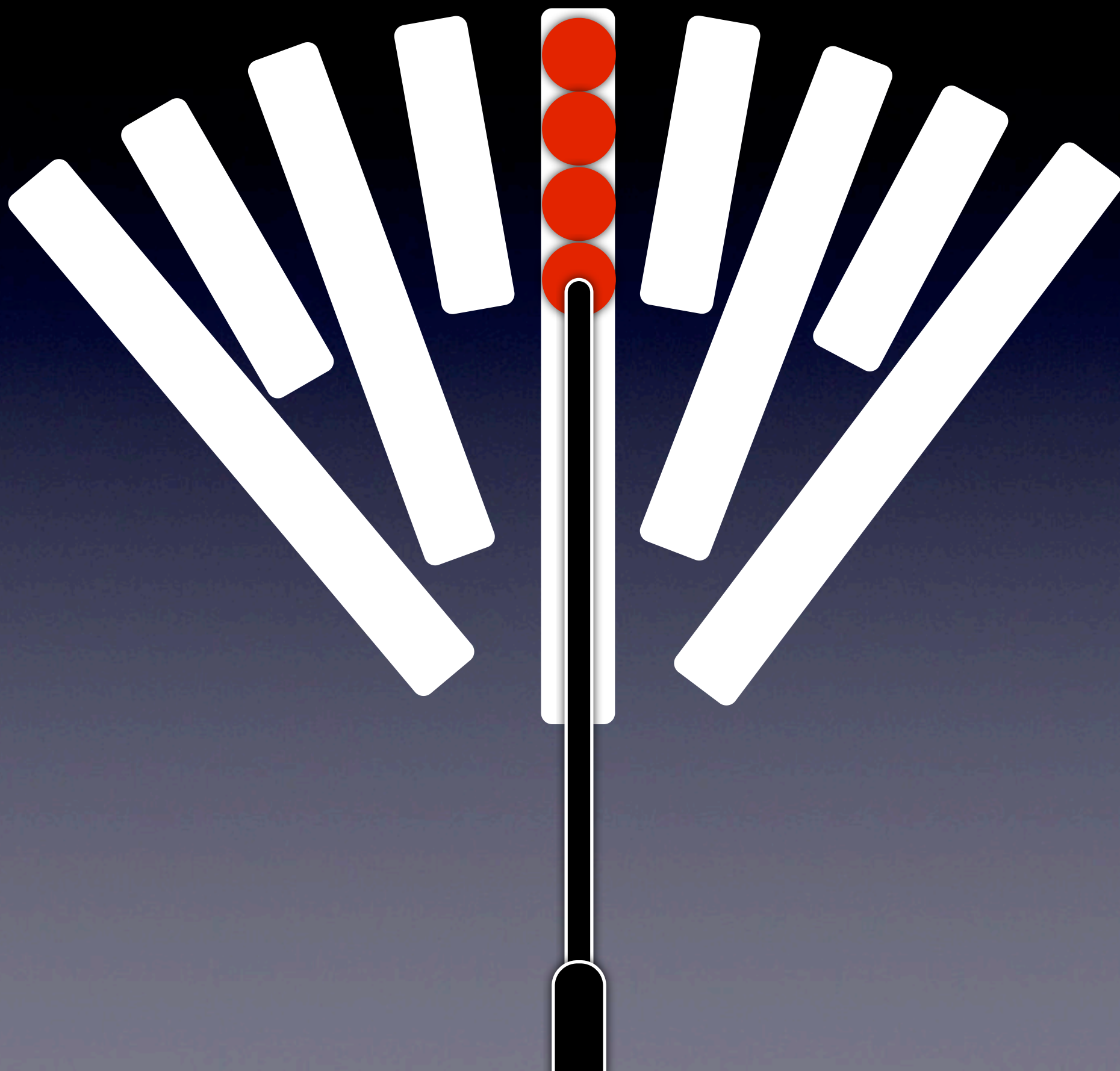


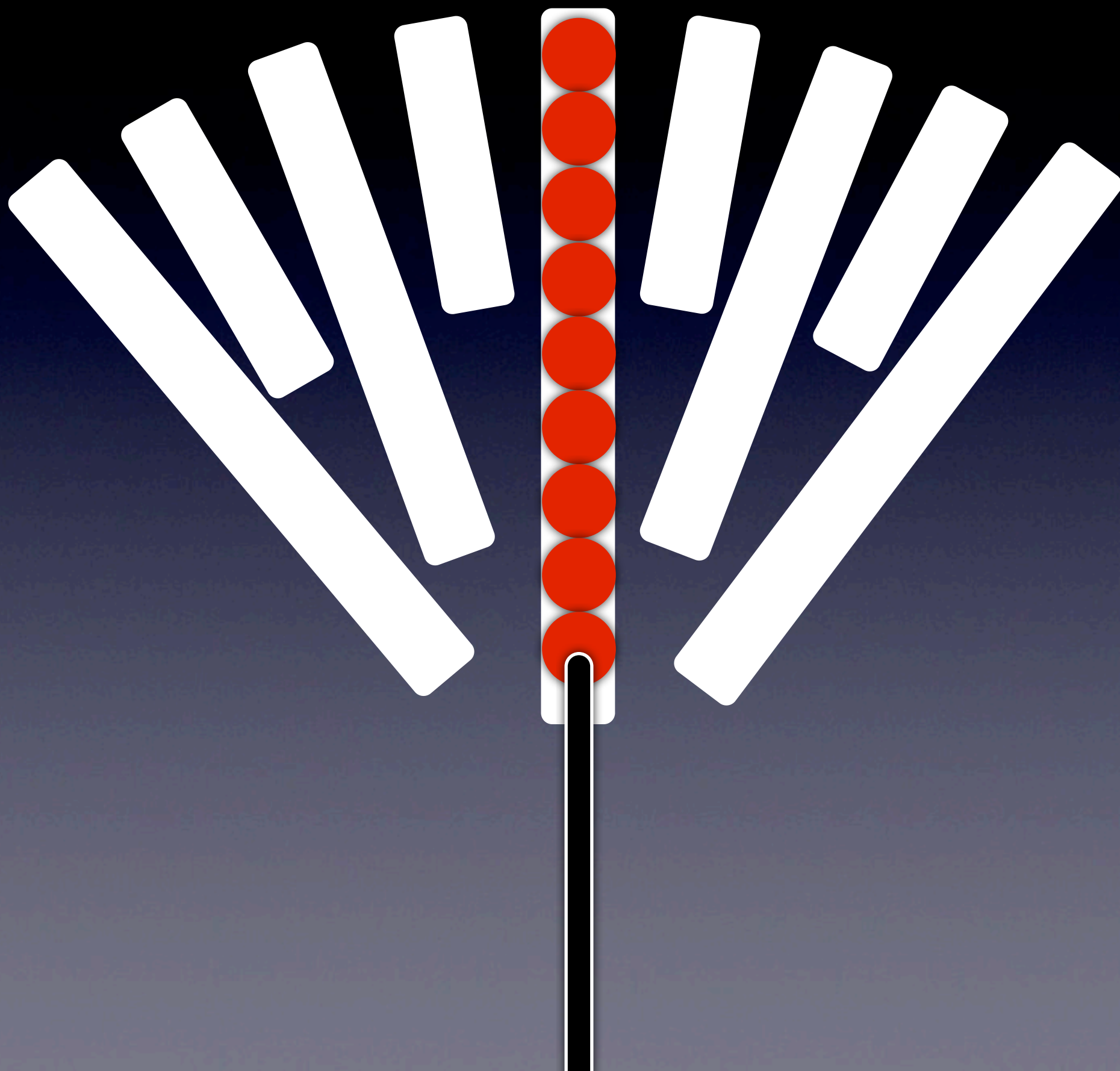

3) Release the ESU switch
before removing the
StarLipo through the
liposuction incision

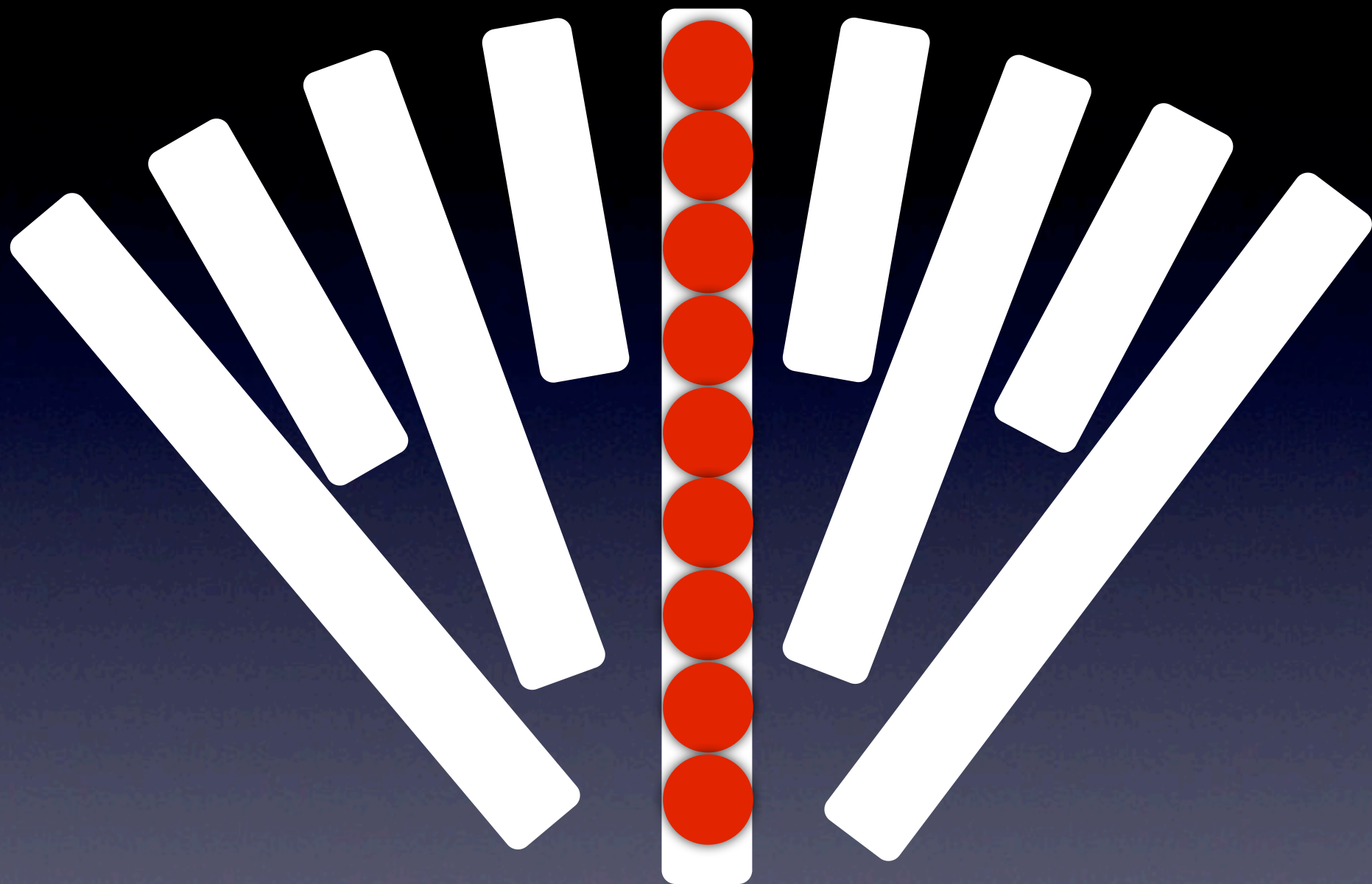

4) Repeat this sequence for the entire area. Single pass by tunnel.

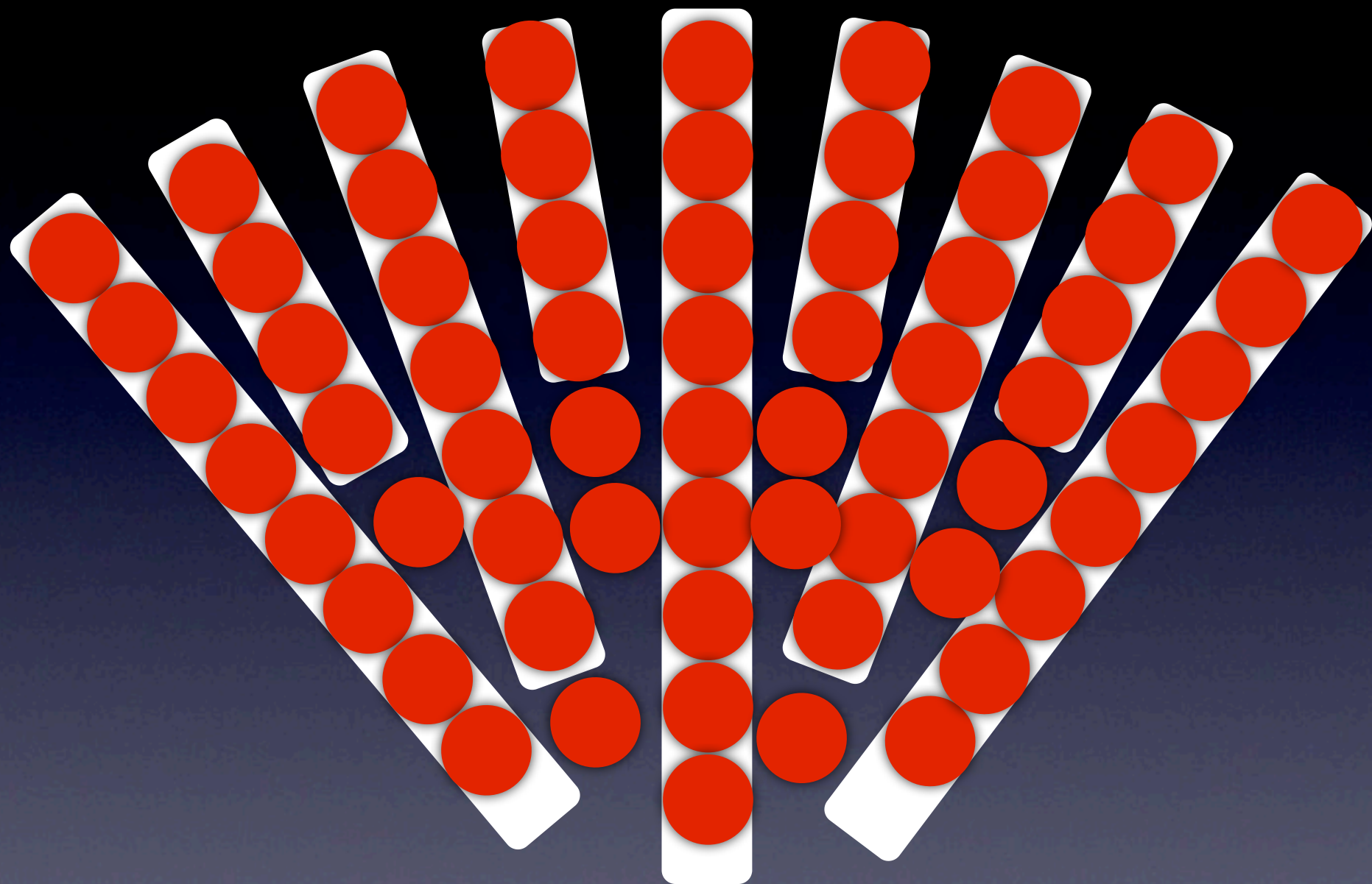

The skin contraction effect starts immediately after the application of the monopolar RF energy. The maximum effect is achieved 6 months after surgery.

The skin contraction effect
can be seen intraoperatively.
For comparison, the StarLipo
procedure is completed at
one side



★ StarLipo

No skin burns

Precise RF dose

Small, lightweight

Zero maintenance

Extremely easy-to-use

★ StarLipo

Reliable skin contraction

Compatible

No learning curve

Savings in OR expenses

Miniaturization technology

FDA Class 1

Ready to use in clinical settings

The StarLipo device is a high-tech device, and should be handle with care accordingly.

Do not use the StarLipo device if either the electrical insulation or the cable are damaged.



The Safest RF Lipoplasty

© 2010-2015 ★ StarLipo. All rights reserved